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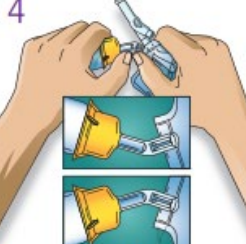
MINI-BAG PLUS system directions:
Only for Single Dose Powdered Drug Vials with 20 mm Closures
Use Aseptic Technique

<p>1</p> 	<p>2</p> 	<p>Assembly</p> <ol style="list-style-type: none"> ① • Remove vial cover • Disinfect stopper ② • Peel off foil cover • Inspect adaptor for moisture Discard if found ③ • Place vial upright • Hold firmly • Push adaptor down until vial snaps in place • DO NOT TWIST • Pull vial to ensure fully seated
<p>3</p> 	<p>4</p> 	<p>Reconstitution</p> <ol style="list-style-type: none"> ④ • Squeeze bag and check vial • Use only if vial fully seated and dry • Bend up then down to break seal ⑤ • Hold bag with vial down • Squeeze solution into vial until half full • Shake to suspend drug in solution ⑥ • Hold bag with vial upside down • Squeeze bag to force air into vial • Release to drain suspended drug from vial • Repeat steps 5 and 6 until vial is empty of drug and solution is thoroughly mixed. Ensure drug is completely dissolved. Do Not Remove Drug Vial.
<p>5</p> 	<p>6</p> 	<ol style="list-style-type: none"> ⑦ • Remove port protector. • Attach administration set per its directions. ⑧ • Hang container on I.V. pole and prime set per directions. Ensure that vial is empty of drug and solution. Repeat step 6 if drug and solution remain in vial. Warning: Do not use in series connections ⑨ • Administer medication per directions. Use within specified time for drug stability. Refer to drug package insert.

Rx only.
 To ensure safe and proper use of this device, refer to the complete instructions for use.

To Order Baxter Products, call 1-888-229-0001

Baxter, Mini-Bag Plus and Committed to a Safer Healthcare Environment are trademarks of Baxter International Inc.
 Baxter Healthcare Corporation, Medication Delivery, Route 120 and Wilson Road, Round Lake, IL 60073 www.baxter.com 111094 5K 04/09

Options at work.

MEDICATION DELIVERY

FIGURA 1
 *ver pasos 4-6

Irving: 8200 Tristar Drive, Suite 120
 Texas 75063

Irving,

Fort Worth: 3515 NW Jim Wright Freeway
 Fort Worth, Texas 76106

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